



**ZDL, Inc.**  
**Zavos Diagnostic Laboratories, Inc.**

**E-mail: [info@zdlinc.com](mailto:info@zdlinc.com)**  
**Please visit our website: [www.zdlinc.com](http://www.zdlinc.com)**

## **The Zavos Home Conception Pak**

### **Instructions for Use\***

*\*If you do not understand these instructions or if you have any questions, please call us at 859-278-6806, or log on to our website at [www.homeconceptionstore.com](http://www.homeconceptionstore.com) and click Contact Us to send us an email.*

### **General Warnings and Cautions**

1. Read and follow instructions in the Zavos Home Conception Pak before use.
2. Use of the Zavos Home Conception Pak is intended to increase the likelihood of becoming pregnant. Pregnancy may involve serious health risks for both you and your unborn child. Consult your healthcare professional for possible health risks before use of the Zavos Home Conception Pak.
3. The Zavos Home Conception Pak components will not protect against HIV or any other sexually transmitted diseases (STD's)
4. To avoid choking hazard, keep all Zavos Home Conception Pak components out of the reach of children.
5. The Zavos Home Conception Pak components are not intended for contraceptive use. Safety and effectiveness of the Zavos Home Conception Pak's Cervical Cup with Stem for contraception have not been established.
6. Do not reuse any Zavos Home Conception Pak component.
7. Do not use any component of the Zavos Home Conception Pak after the expiration date.
8. The Zavos Home Conception Pak can be stored at temperatures of between 2° and 30° Celsius or 35° to 86° Fahrenheit. The Zavos Home Conception Pak should be kept away from direct sunlight, moisture, and heat.